

TRIDA Hydro Colloid Dressing



Effectively isolate and absorb exudate makes the healing naturally.



**Patented
3D waved
design**

*Soft and waterproof material attached to wound smoothly,
and provides gentle protection.*

TRIDA Company Limited

1F., No.6, Qingnian Rd., Youth Industrial Park, Yangmei Dist.,
Taoyuan City 32661, Taiwan (R.O.C)

Tel: +886 3 464 2482 Fax: +886 3 4641432

Email: service@igiban.com Website: www.igiban.com

品質系統
認證
Certification



厚生労働省
ひと、くらし、みらいのために
外国製造業者登録証





1. Concept:

The hydrocolloid dressing is a kind of advanced Moist wound dressings and is mainly composed of hydrophilic colloid, adhesives, PU film.

2. Principle:


Covering moist dressings on the wound to absorb exudate and make the hydrophilic colloid swell and changed to white and formed into a gel. Maintain the surface of wound in a moisture environment and do not scab. laticity.

3. Characteristics:

1. Contains with variety of sizes, shapes (3 dimensions and flat), specially designed for the needs of different parts of the body.
2. Soft texture of materials can be closely fit the affected area and effectively isolate the external environment.
3. Absorbs exudate of the wound rapidly, make the hydrophilic colloid swell and changed to white. Maintain the surface of wound in a moisture environment.
4. Waterproof outer layer provides comprehensive protection for the wound when taking shower and bath.
5. Contains with elastic characteristic, can be attached on the wound smoothly and doesn't affect your daily life.
6. Can be continuously use if there is no leakage, peeling, polluted or uncomfortable situation.
7. Easy to remove and won't remain adhesive residue on wound. Do not cause secondary damage and it's not painful when replace.
8. No scab on the wound and recover beautifully.

4. Applications:

1. All kinds of shallow, middle depth of wound with exudate
2. All kinds of superficial skin injury (such as: rub down, tear, cuts)
3. A small area of shallow burns
4. Surgical wounds
5. The wound after laser beauty treatment
6. Protection for friction (such as: new shoes foot wear, blister prevention).

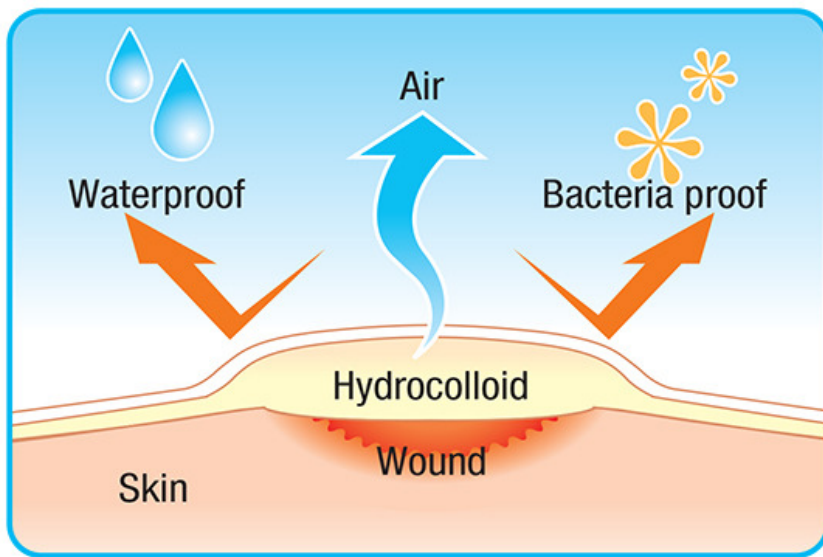

Three rules of healing by moisture:



Merits of Moist Healing

- 1 Faster healing of wounds.**
- 2 Lessen pains.**
- 3 Less scars.**


Dry Healing

- They allow exposure of the wound to the air. Covers the wound with cotton pad to dry the blood and let scabs happen.
- With decreased body fluid, the wound cannot remain in moist condition. Recovery takes longer.
- Scabs make it impossible for cells to move fast, grow and become epidermis.



Moist Healing

- The covering with moisturizing plaster isolates the air and allows the wound to keep moist.
- Effectively keeps moist and accelerates recovery of the wound.
- Retains body fluid and allows the cells to mobilize fast. Speeds up transformation of cells into epidermis.

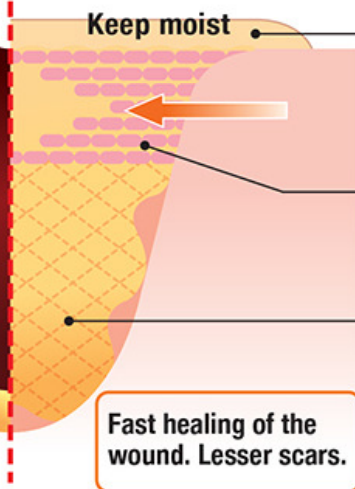

Comparison of dry and wet cases



**IGIBAN**

3D Hydro Colloid Plasters

Indications: Limbs/bending parts/wounds of small area
Small blisters. Minor cuts. Burns. Laser wounds. Hand eczema. Bug bites

Features:


- Clear, waterproof and one-piece design
- Patented wavy design. Thick center and thin edges. Fits well and comfortable. sticks fast.




- Item No.: 3660624
- Specification: 2x7 cm
- Small package: 10 pcs/pack
- Medium package: 10 packs/box
- Large package: 100 packs/carton



- Item No.: 3660631
- Specification: 3x7 cm
- Small package: 6 pcs/pack
- Medium package: 10 packs/box
- Large package: 100 packs/carton

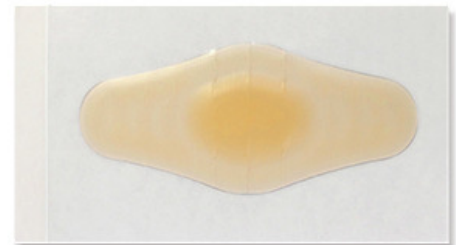

- Item No.: 3660648
- Specification: 4.5x7 cm
- Small package: 5 pcs/pack
- Medium package: 10 packs/box
- Large package: 100 packs/carton

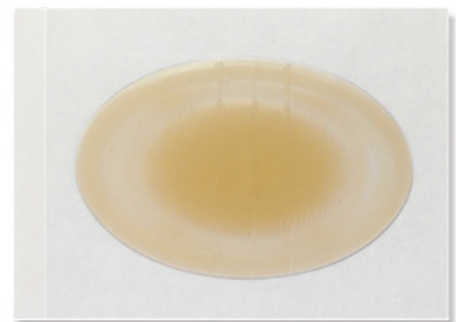**IGIBAN**

T⁺ IGIBAN 3D Hydro Colloid Pad

Indications: Small-area superficial wounds
Nail edges. Acne. Laser wounds. Bug bites.

Features:

- Clear. Waterproof. One-piece design
- Patented design with single-hand operation. Thick center and thin edges. Fits well and comfortable. sticks fast.



- Item No.: 3660686
- Specification: 1.2x1.5 cm
- Small package: 30 pellets/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton



- Item No.: 3660662
- Specification: 1.3x1.3 cm
- Small package: 24 pellets/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton



Four steps of moisturizing and healing the wound:

1. Rinse:
rinse the wound with water.



2. Wipe:
carefully examine and wipe off the moisture from the wound.



3. Apply:
apply the moisturizing dressing to the wound to cover it completely.



4. Watch:
observe carefully how the wound is healing.



**IGIBAN**

Hydro Colloid Dressing

Indications: Fall wounds and scratches; superficial burns; wounds with more exudate (body fluid)

Features:

- Clear and waterproof.
- Colloid absorption and body fluid retention



- Item No.: 3661225
- Specification: 5x7 cm
- Small package: 4 pcs/pack
- Large package: 60 packs/carton



- Item No.: 3661218
- Specification: 6x8 cm
- Small package: 3 pcs/pack
- Large package: 60 packs/carton



- Item No.: 3661201
- Specification: 8x10 cm
- Small package: 2 pcs/pack
- Large package: 40 packs/carton

**IGIBAN**

**IGIBAN**

Hydro Colloid Dressing



- Item No.: 3661195
- Specification: 10x10 cm
- Small package: 1 pcs/sachet



- Item No.: 3661188
- Specification: 5x20 cm
- Small package: 2 pcs/sachet



- Item No.: 3661171
- Specification: 5x10 cm
- Small package: 3 pcs/sachet



**IGIBAN**

Hydro Colloid Foot Plaster


Indications: Prevention in wearing shoes; abrasion cushioning; softening callosity; prevents dry and cracked skin

Features:

- Clear; waterproof; one-piece design
- Patented wavy design; Thick center and thin edges; Fits well and comfortable; sticks fast.
- Prolonged use can soften keratin and prevent cracks from drying.



- Item No.: 3660150
- Specification: 2x6 cm
- Small package: 8 pcs/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton

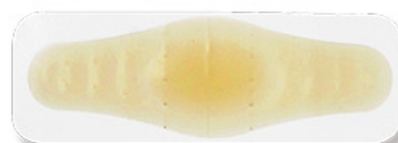

- Item No.: 3660167
- Specification: 3.6x5.6 cm
- Small package: 6 pcs/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton



- Item No.: 3660204
- Specification: 3.6x6 cm
- Small package: 6 pcs/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton



- Item No.: 3660280
- Specification: Assorted
- Small package: 8 pcs/pack
- Medium package: 15 packs/box
- Large package: 150 packs/carton



1703-A0

IGIBAN